



SOMA SAFE ENCLOSURE - ACCESSORY PORTS

There are multiple access ports to allow IV's, Foleys, air mattress support systems, vents, call buttons, remotes, etc. to be placed for patient care, comfort, and safety. All access ports are zipper controlled with access only from the outside.

There are four IV or call button access ports, two on each side. One is located near the patient's wrist and the other is located adjacent to the vertical collar on the side panel at the head end of the Soma Safe Enclosure. To use, un-zip or open the zipper and place the device through the opening. Close the zipper part way to secure the line in place. Be careful not to crimp or pinch the line.

There are two Foley ports one on each side adjacent to the patient's lower abdomen. These openings are wider than the IV ports to allow the bag to pass through without the need to disconnect the tubing. Close the port zipper when using but not to the point of crimping or pinching the line.

There are two additional openings, one at the head end and one at the foot end. Both are located at the bottom of the zippered panel. These ports can be used for additional lines, tubing or air-mattress connections as needed.

The bottom of the net has a variety of openings that allow connection to various mattresses and support surfaces. Be sure to keep all unused ports zipped closed when not in use.

